

Vitalograph 4000 Series Products Used as a Personal Home Monitor Care and Cleaning of the Respiratory Monitor

The Respiratory Monitor should continue to give reliable measurements for up to three years in home use. Then replace it with a new device.

Keep it clean and dust free. If you suspect the device is damaged or is measuring incorrectly, contact the doctor immediately.

The outer surfaces should be thoroughly cleaned every week, more often if necessary. We recommend the use of an ordinary alcohol wipe, paying special attention to the mouthpiece area.

Part	Material	Cleaning Recommendation	Disinfection Recommendation
Plastic Mouthpiece	ABS	Wash in warm soapy water. Rinse in clean water	Alcohol wipe (IPA 70-90%)
Body	ABS	Wipe with a damp cloth	Alcohol wipe (IPA 70-90%)
Fascia	PMMA/PET	Wipe with a damp cloth	Alcohol wipe (IPA 70-90%)
Buttons	Synthetic Rubber	Wipe with a damp cloth	Alcohol wipe (IPA 70-90%)

In cleaning the body to remove visible particulate contamination, take care not to touch the moving vanes inside the device.

Vitalograph Ltd . Maids Moreton . Buckingham . MK18 1SW . England . Tel: +44 (0) 1280 827110 .
 Fax: +44 (0) 1280 823302. E-mail: sales@vitalograph.co.uk www.vitalograph.co.uk
 Vitalograph GmbH . Jacobsenweg 12 . 22525 Hamburg . Germany . Tel: (040) 547391-0 . Fax: (040) 547391-40. E-mail: info@vitalograph.de www.vitalograph.de
 Vitalograph Inc . 13310 W. 99th St . Lenexa . Kansas 66215 . USA . Tel: (913) 888 4221 . Fax: (913) 888 4259. E-mail: vitcs@vitalograph.com www.vitalograph.com
 Vitalograph (Ireland) Ltd . Gort Road Business Park. Ennis . Co Clare . Ireland. Tel: (065) 6864100 .
 Fax: (065) 6828289. E-mail: sales@vitalograph.ie www.vitalograph.com

©Vitalograph®, SafeTway®, BVF™ are trademarks of Vitalograph

07385_5